Instruction for Use

- 1 **[Family name]** GP Remover
- 2 **[Manufacturer]** YDM CORPORATION
- 3 **[Model No and Device trade name]**

Model No.	Device trade name	UDI-DI	Working part diameter
27131	GP Remover Spear 0.3S	4963931271313	0,3 mm
27132	GP Remover Spear 0.3L	4963931271320	0,3 mm
27133	GP Remover Spear 0.5S	4963931271337	0,5 mm
27134	GP Remover Spear 0.5L	4963931271344	0,5 mm
27135	GP Remover Spear 0.7S	4963931271351	0,7 mm
27136	GP Remover Spear 0.7L	4963931271368	0,7 mm

4 **[The device's intended purpose]**

The device is used to explore and remove gutta-percha and other debris from the root canal walls in re-treatment.

5 **[The performance characteristics of the device]**

<Overview>

The device is made of one piece, and the working end has a shape effective to serve its intended purpose.

<Material>

Stainless steel

6 **[How to use]**

- 1) Attach to one of the following handles.

Model No.	Device trade name	UDI-DI	Material
10201	P-A Handle	4963931102013	Stainless steel and Brass
10202	P-B Handle	4963931102020	
10203	P-C Handle	4963931102037	Stainless steel and Brass
10205	P-F Handle	4963931102051	

- 2) Explore and remove gutta-percha and other debris from the root canal.

7 **[General precautions and prohibition]**

- 1) The use of the product is strictly limited to dental treatments.
Precautions must be followed to ensure the appropriate use of the instruments
- 2) Do not use the instruments for anything other than its intended use.
- 3) Clean, lubricate and sterilize before using instructed by IFU.
- 4) Do not use the instruments for patients who have a disease of hypersensitivity symptoms such as rash and dermatitis or allergic symptoms.
- 5) Do not change the design or remodel the instruments
- 6) Do not drop the instruments. Avoid any strong impact on the instruments.
- 7) When disposing of the instruments, follow the instructions detailed in appropriate national regulations.
- 8) Do not heat the instruments.
- 9) If chemicals adhere to the instruments, wipe it off immediately to avoid corrosion.
- 10) Stainless steel is more resistant to rust than iron, however it may corrode due to method of use and environment like temperature, humidity.

8 **[Limitations on processing]**

Stop using the instruments immediately if you find out deterioration or something strange.

9 **[Initial treatment at the point of use]**

Clean, lubricate and sterilize before using as IFU instructed.

10 **[Preparation before cleaning]**

See next section.

11 [The process of cleaning and sterilization/disinfection]

1) General cautions

- a) In case of maintenance of instruments, please refer to the "The Fundamental Guide Book for Cleaning and Sterilizing Instruments" published by our company or the item of "maintenance" on our website (http://www.ydm.co.jp/en-new/pdf/The_process_of_cleaning_and_sterilizing.pdf).
- b) The instruments contaminated with blood, body fluid, tissue fragments, chemicals, etc. shall be cleaned and sterilized immediately in compliance with following method in order to prevent the contaminants from drying out and sticking.

Contaminants attached to the instruments for a long time become difficult to be removed.

2) Procedure

- a) Separate the instruments and the handle without delay after examining/treating each patient.
 - b) Clean the instruments with a cleaning agent in the method of i) or ii).
 - i) Use an ultrasonic cleaner with medical rust preventive detergent for 5minutes.
 - ii) Immerse in a cleaner with a rust preventive detergent for medical use for 15 minutes.
 - c) Wash off dirt and cleaner adhered to the instruments with running water.
(If soap components remain on the surface of instruments, it cause rust,stains.)
 - d) Dry the instruments. (Remaining moisture may cause rust,reduce sterilization effect)
 - e)Sterilize with using an autoclave sterilizer.
Conditions: 134°C to 138°C, 3 minutes to 3.5 minutes
- * To prevent scratches because of contact with other instruments, put them in a sterilized pack or case.

f) After use, clean immediately according to b) above.

3) Cautions of cleaning and sterilization/disinfection

- a) Do not use the following chemicals as they may cause metal corrosion.
sodium hypochlorite, benzalkonium chloride, chlorhexidine gluconate, tincture of iodine, Iodoform and peracetic acid may corrode metal.
- b) Do not use chemical autoclave on the instruments, which may cause corrosion.
- c) Do not perform plasma sterilization as it may affect the material.
- d) Do not use household cleaning agents as it's not suitable with remove protein on the surface of instruments.
- e) Wire brush and steel wool may cause damage instruments.
- f) Do not use Super Acidic Water as it may cause corrosion.
Purified water (no impurities) is recommended for rinsing.
Tap water contains chlorine,which may corrode the metal.
- g) When using a washer-disinfector, the stucked dirt or cleaning solution may not be completely removed if water saving or a time saving program is used. Be sure to refer to the manufacturer's instruction manual, etc. to ensure rinsing and to remove dirt.
- h) Note the following when handling autoclave sterilizers.
 - i) The autoclave sterilizer may be contaminated in the cabinet depending on the usage status and period. If dirt remain autoclave sterilizer they may stick to the instruments.
We recommend cleaning according to instruction manual of the sterilizer.
Then inside of the chamber does not become dirty. In particular, the rubber seal on the chamber lid and air filter shall be replaced in a certain of period.
 - ii) Use purified water as much as possible. When tap water may cause of corrosion due to the influence of chlorine ions.
 - iii) Pay attention to the drying temperature inside autoclave sterilizer.
 - iv) Do not place this instruments near the heater. (It may be higher than the indicated temperature)
 - v) If cleaning and rinsing are not complete inside autoclave sterilizer, or moisture remains autoclave sterilizer chamber, it may cause of burn on the surface of the instruments.
- i) After washing and sterilization, remove the water adhering to the instrument and dry it completely

12 [Maintenance, Inspection and Testing]

After cleaning and sterilization, check the following before use.

- 1) Check that there is no dirt, damage, cracks, scratches.

13 [Storage]

- 1) Store in a clean place free from dust and dust, avoiding high temperature and humidity.
- 2) To prevent galvanic corrosion due to metal potential difference, do not store instruments of different materials.
- 3) Note the following in order to prevent rust stains.
 - a) Do not store with rusted instruments.
 - b) Do not store with chemicals.
 - c) Pay attention to rust inside sterilizers and storages.

- 4) Carefully move and store in a tray / container as this may cause deformation.
When using trays or containers, do not place heavy equipment.

14 [Disposal considerations]

Dental specialists and / or orthodontists are responsible for disposal in accordance with relevant applicable law.
YDM CORPORATION doesn't take responsibility for disclaims caused by improper disposal personal injury.

15 [Additional information]

<Breakage>

In case of breakage of instruments, please report to dealer from which you purchased in format prepared by YDM CORPORATION named "Claim Report".

16 [Manufacturer contact]

YDM CORPORATION

28 Imaizumi, Higashi-Matsuyama-shi,
Saitama 355-0042, Japan

TEL: 81-3-3828-3161 FAX: 81-3-3827-8991
URL: <http://www.ydm.co.jp/> E-mail: ydm@ydm.co.jp

17 [Notice to the user and/or patient]

Any serious incident that has occurred in relation to the instruments should be reported to the YDM CORPORATION.

18 [Additional information]

In case of defect of device, please report to dealer from which you purchased in format prepared by YDM Corporation named "Claim Report".

19 [Description of symbols]

Medical device

Manufacturer

Lot number

Notice

The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

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