Instruction for Use

1 [Family name] Retractors

2 [Manufacturer] YDM CORPORATION

3 [Model No and Device trade name]

Model No.	Device trade name	UDI-DI	Material
14620	Cheek Retractor	4963931146208	Stainless steel
14657	Cheek Retractor Stemberg L	4963931146574	Stainless steel
14658	Cheek Retractor Stemberg S	4963931146581	Stainless steel
14661	Tongue Depressor	4963931146611	Stainless steel
14676	Mouth Prop with Tongue Guard L	4963931146765	(Mouth Prop) Silicon rubber (Tongue Guard) Polypropylene
14677	Mouth Prop with Tongue Guard S	4963931146772	
14678	Mouth Prop with Tongue Guard SS	4963931146789	
22721	Deep Retractor for Bi-Lateral #1	4963931227211	Stainless steel
22722	Deep Retractor for Bi-Lateral #2	4963931227228	
22723	Deep Retractor SU Adult	4963931227235	
22724	Deep Retractor SU Child	4963931227242	
22725	Cheek Retractor Doublewider SS (2 ea.)	4963931227259	
22726	Cheek Retractor Anglewider One Hand (2 ea.)	4963931227266	Polypropylene
22727	Cheek Retractor Anglewider Wide (2 ea.)	4963931227273	TPX
22728	Cheek Retractor Anglewider for Child (2 ea.)	4963931227280	
22729	Cheek Retractor Anglewider Soft (2 ea.)	4963931227297	
22731	Cheek Retractor A (2 ea.)	4963931227310	
22732	Cheek Retractor B (2 ea.)	4963931227327	
22733	Cheek Retractor C (2 ea.)	4963931227334	
22734	Cheek Retractor D (2 ea.)	4963931227341	
22739	Cheek Retractor Pigeon M	4963931227396	
22740	Cheek Retractor Pigeon S	4963931227402	
22741	Cheek Retractor Pigeon SS	4963931227419	
22745	Cheek Retractor Anglewider (2 ea.)	4963931227457	Polypropylene
22749	Cheek Retractor Doublewider S (2 ea.)	4963931227495	
22750	Cheek Retractor Doublewider L (2 ea.)	4963931227501	

4 [The device's intended purpose]

Use to retract the lips/cheeks of a patient during an orthodontic or restorative dental procedure.

5 [The performance characteristics of the device]

<Overview>

This device has a variety of shapes to retract the lips/cheeks.

<Material>

Refer to "3 [Model No and Device trade name]"


6 [How to use]

Model No.	Device trade name	How to use
14620	Cheek Retractor	Retraction of corners of the mouth or lips / Traction of tissues such as gums
14657	Cheek Retractor Stemberg L	Retract the corners of the mouth
14658	Cheek Retractor Stemberg S	
14661	Tongue Depressor	Retract the tongue and cheeks, and keep the mouth open
14676	Mouth Prop with Tongue Guard L	See a) below
14677	Mouth Prop with Tongue Guard S	
14678	Mouth Prop with Tongue Guard SS	

22721	Deep Retractor for Bi-Lateral #1	Keep the mouth open by retracting the corners of the mouth, lips, buccal mucosa, etc.
22722	Deep Retractor for Bi-Lateral #2	
22723	Deep Retractor SU Adult	
22724	Deep Retractor SU Child	
22725	Cheek Retractor Doublewider SS (2 ea.)	See b) below
22726	Cheek Retractor Anglewider One Hand (2 ea.)	See c) below
22727	Cheek Retractor Anglewider Wide (2 ea.)	See d) below
22728	Cheek Retractor Anglewider for Child (2 ea.)	
22729	Cheek Retractor Anglewider Soft (2 ea.)	
22731	Cheek Retractor A (2 ea.)	
22732	Cheek Retractor B (2 ea.)	Keep the mouth open by retracting the corners of the mouth, lips
22733	Cheek Retractor C (2 ea.)	
22734	Cheek Retractor D (2 ea.)	
22739	Cheek Retractor Pigeon M	
22740	Cheek Retractor Pigeon S	
22741	Cheek Retractor Pigeon SS	
22745	Cheek Retractor Anglewider (2 ea.)	See c) below
22749	Cheek Retractor Doublewider S (2 ea.)	See b) below
22750	Cheek Retractor Doublewider L (2 ea.)	

a) Mouth Prop with Tongue Guard


- (1) When making the patient bite the mouth prop on the right side and controlling the left side of the tongue, insert tongue guard L into the mouth prop.

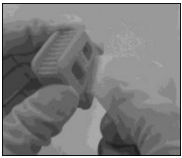
- (2) Let the patient bite at right posterior side and adjust the position of the tongue guard to better control the tongue.


- (3) Lastly, check to see whether the patient is firmly biting on the mouth prop.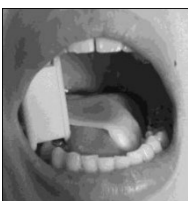


b) Cheek Retractor Doublewider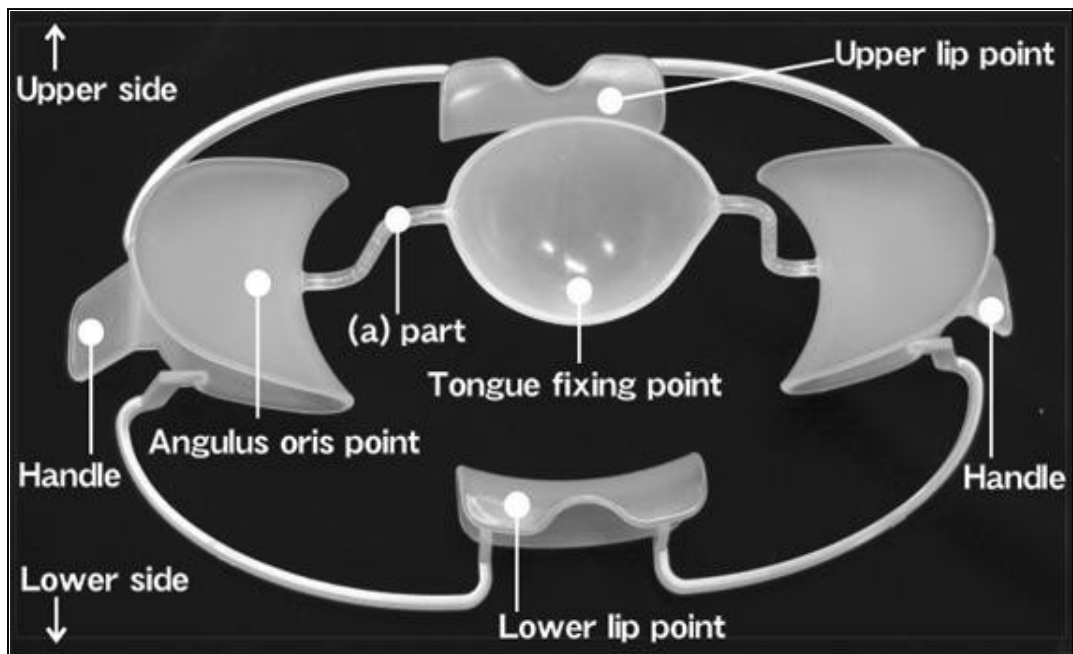


(1) Make sure the up and down by the arrow on the tongue holding part.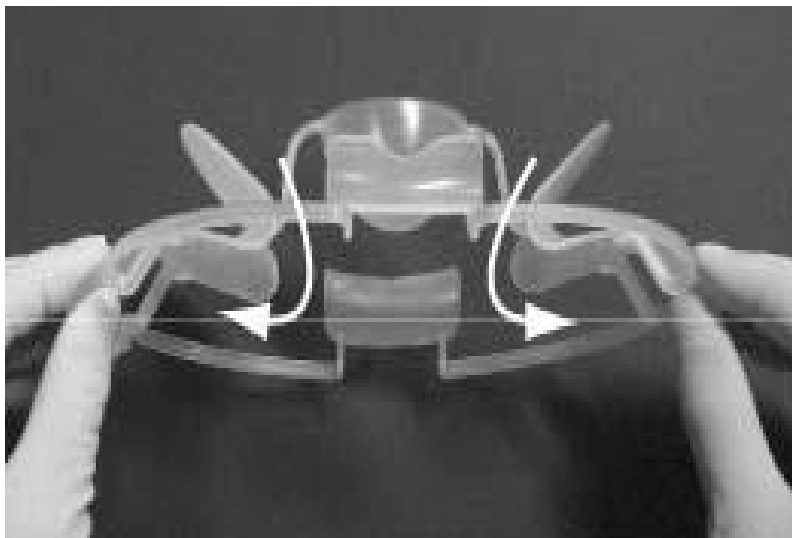

(2) Twist both handle parts.

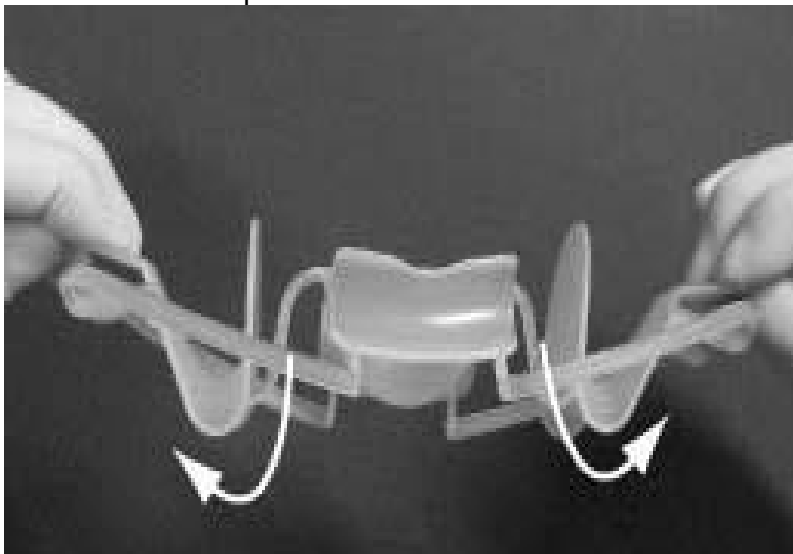
(3) Insert the right and left cheek parts into the mouth.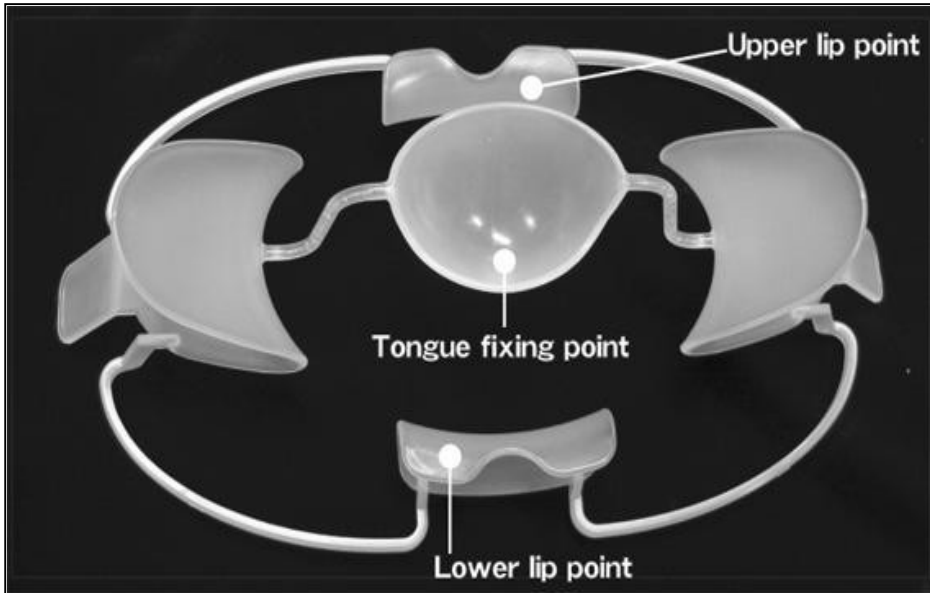


(4) Place lip parts to the upper and lower lips.

c) Cheek Retractor Anglewider One Hand
<View from the front>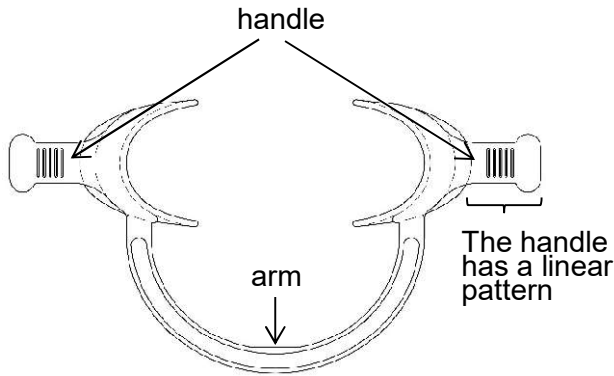

<View from above>

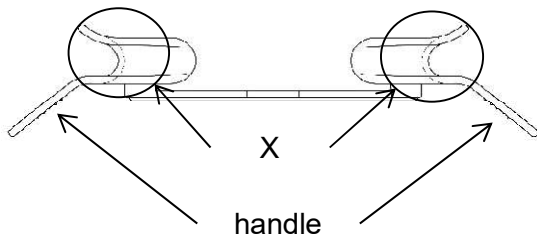
X:Area to be applied to the corner of the mouth

① Grasp the arm with one hand, facing the linear pattern, and bend it slightly (flexing it) as if you were "creating a mountain fold from the center of the arm".

NOTE:When shrinking this product, if you move the arm in the direction shown in the figure below, it may break.

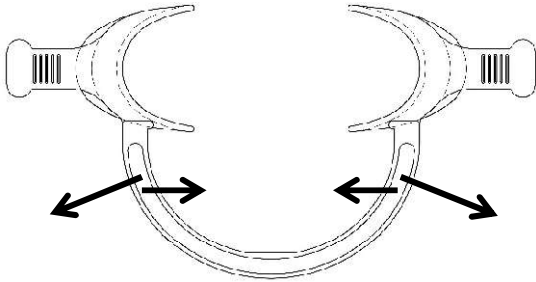
② Insert the lips into the X, one on each side.


③ Slowly release the bend and adjust the position using the handle.

d) Cheek Retractor Anglewider
<View from the front>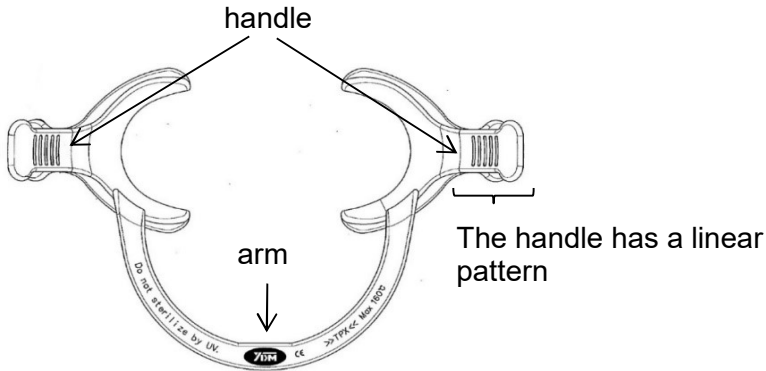


<View from above>


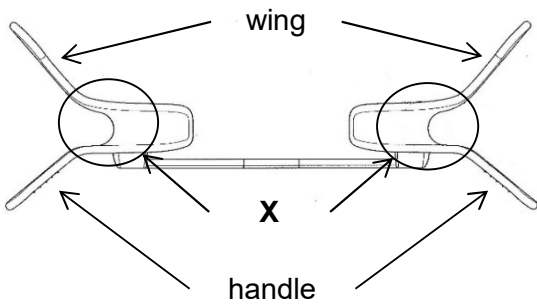
X:Area to be applied to the corner of the mouth

① Hold the handles and shrink it in the direction of the arrow.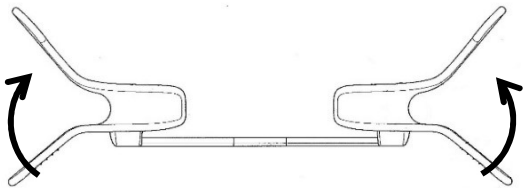

NOTE:When shrinking this product, if you move the arm in the direction shown in the figure below, it may break.

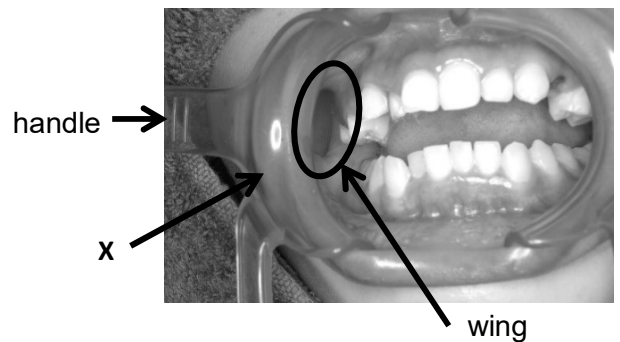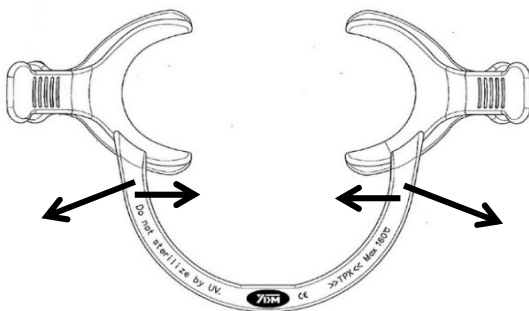- ② Insert the wings on both sides along the buccal mucosa into the oral vestibule.
- ③ Insert the lips into **X** in the order of upper lip and lower lip.
- ④ Hold the handles on both sides and gradually expand the arm while excluding the buccal mucosa with the wings.

7 [General precautions and prohibition]

- 1) The use of the product is strictly limited to orthodontic or dental treatments.
Precautions must be followed to ensure the appropriate use of the product.
- 2) Do not use the product for anything other than its intended use. The use of the product is limited to dental professionals.
The use of the product is limited to dental professionals.
- 3) Properly clean, lubricate and sterilize before sterilization using the method and conditions specified by IFU for each patient.
- 4) Patients who have developed hypersensitivity symptoms such as rash and dermatitis or allergic symptoms due to treatment with this device should be discontinued and have a doctor diagnosed.
- 5) Do not change the design or remodel the product.
- 6) Do not drop the product. Avoid any strong impact on the product.
- 7) When disposing of the product, follow the instructions detailed in appropriate national regulations.
- 8) Do not heat product.
- 9) If chemicals adhere to the product, wipe it off immediately as it may corrode.
- 10) Stainless steel is more resistant to rust than iron, however it may corrode due to method of use and environment like temperature, humidity.

8 [Limitations on processing]

Stop using the product immediately if deterioration or abnormality are visible.

9 [Initial treatment at the point of use]

Properly clean, lubricate and sterilize before sterilization using the method and conditions specified by IFU for each patient.

[Preparation before cleaning]

See next section.

11 [The process of cleaning and sterilization/disinfection]

- 1) General cautions
 - a) When processing this product, please refer to the "The Fundamental Guide Book for Cleaning and Sterilizing Instruments" published by our company or the item of "maintenance" on our website (<http://www.ydm.co.jp/>).
 - b) The device contaminated with blood, body fluid, tissue fragments, chemicals, etc. shall be cleaned and sterilized immediately in compliance with following method in order to prevent The contaminants from drying out and sticking. Leaving The contaminants attached can make it difficult to remove.
- 2) Procedure
 - a) **(If applicable)** Open the joint.
 - b) Clean the device with a cleaning agent in the method of i) or ii).
 - i) Using an ultrasonic cleaner with medical rust preventive cleaner [for example, "Z-1 eco (sold separately)"] is dissolved in warm water, and ultrasonic cleaning is performed for 5 minutes.
 - ii) Immerse and wash in a rust preventive detergent for medical use for 15 minutes.
 - c) Wash off dirt and cleaning liquid adhered to the device with running water. (If soap components remain in the cleaning solution and remain attached, it may cause rust and stains.)
 - d) Allow the device to dry. (Remaining moisture may cause rust and reduced sterilization effect)
 - e) **(If applicable)** Add anti-rust lubricant such as "instrument oil (sold separately)" to the joint.
* Do not open or close until anti-rust lubricant is added. (Opening and closing without oil will cause galling)
 - f) Sterilize using an autoclave sterilizer.
Conditions: 134°C to 138°C, 3 minutes to 3.5 minutes
* To prevent scratches due to contact with other devices, it is recommended to put them in a sterilized pack or case.

(Only to 22725 / 22749 / 22750 Cheek Retractor Doublewider SS / S / L)

Perform one of the following x) and y).

x) Immersion cleaning

- Immerse in "glutaral preparation (glutaraldehyde) 2% solution".
- Wash thoroughly with a large amount of purified water.
- If the equipment has holes, clean the inside thoroughly.

y) EOG Sterilization

g) After use, Clean immediately according to b) above.

3) Cautions of cleaning and sterilization/disinfection

- Do not use the following chemicals as they may cause metal corrosion.
sodium hypochlorite, benzalkonium chloride, chlorhexidine gluconate, tincture of iodine, Iodoform and peracetic acid may corrode metal.
- Do not perform unsaturated Chemical Vapor (Chemical Autoclave) on the product, as such agents may cause corrosion.
- Do not perform plasma sterilization because it affects the material.
- Do not use household cleaning agents as such chemical may lead to corrosion.
Use of Antirust Cleaner(13-538 Z-1 eco Fine Liquid) for dental instrument is recommended.
- Wire brush and steel wool may cause some damage for instruments.
Do not use such materials when you clean instruments
- Do not use Super Acidic Water as such agents may cause corrosion.
Purified water (no impurities) is recommended for rinsing.
Tap water contains chlorine. This chlorine may corrode the metal.
- When using a cleaning device such as a washer-disinfector, the attached dirt or cleaning solution may not be completely removed if water saving or a time saving program is used. Be sure to refer to the manufacturer's instruction manual, etc. to ensure rinsing and to remove dirt.
- Note the following when handling autoclave sterilizers.
 - The autoclave sterilizer may be contaminated in the cabinet depending on the usage status and period. If autoclave sterilization is performed with dirt attached, stains may adhere to the instrument. Encourage regular cleaning according to the package insert or instruction manual of the sterilizer so that the inside of the chamber does not become dirty. In particular, the chamber lid packing and air filter may require periodic replacement.
 - Use purified water (pure water) as much as possible. When tap water is used, the equipment may corrode due to the influence of chlorine ions.
 - Pay attention to the drying temperature and internal temperature.
 - Do not place this product near the heater. (It may be higher than the indicated temperature)
 - If there is a risk of the inside of the chamber becoming hot, preheat drying. High temperature drying may cause the instrument to be altered or discolored, deteriorated or damaged.
 - If autoclave sterilization is performed while cleaning and rinsing are not complete, or when water is adhering to the inside of the autoclave sterilizer chamber, there is a risk of seizure of the instrument.
- After washing and sterilization, remove the water adhering to this instrument and dry it thoroughly before storing. Leaving it for a long time with moisture attached may cause rusting, spots, etc.

12 **[Maintenance, Inspection and Testing]**

After cleaning and sterilization, check the following before use.

- Check that there is no dirt, damage, cracks, scratches (**If applicable**) or corrosion on the inside of the joint and the joint.
- (**If applicable**) Check for abnormal movement of the joint.

13 **[Storage]**

- Store in a clean place free from dust and dust, avoiding high temperature and humidity.
- To prevent galvanic corrosion due to metal potential difference, do not store instruments of different materials together.
- Note the following in order to prevent rust stains.
 - Do not store with rusted instruments.
 - Do not store with chemicals.
 - Pay attention to rust generated inside sterilizers and storages.

4) Carefully move and store in a tray / container as this may cause deformation. When using trays or containers, do not place heavy equipment.

14 **[Disposal considerations]**

Dental specialists and / or orthodontists are responsible for disposal in accordance with relevant applicable law.

YDM CORPORATION by improper disposal. expressly disclaims any liability for the spread of illness or personal injury caused

15 **[Additional information]**

<Breakage or Sharpening>

In case of breakage of product, please report to dealer from which you purchased in format prepared by YDM CORPORATION named "Claim Report".

Please see more details subject "K".

In case of sharpening of cutters, we will do with a certain of charges.

16 **[Manufacturer contact]**

YDM CORPORATION

28 Imaizumi, Higashi-Matsuyama-shi,
Saitama 355-0042, Japan

TEL: 81-3-3828-3161 FAX: 81-3-3827-8991

URL: <http://www.ydm.co.jp/> E-mail: ydm@ydm.co.jp

17 **[Notice to the user and/or patient]**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

18 **[Additional information]**

In case of defect of device, please report to dealer from which you purchased in format prepared by YDM Corporation named "Claim Report".

19 **[Description of symbols]**

Medical device

Manufacturere

Lot number

Notice

The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

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